

PRODUCT CODE: SIC55DH-HR0500

HEMIA 55 DIFF Lyse Instruction

REF:33001500

【Product name】 Specification: 500mL

Sicuro Bio Name: DIFF Lyse

【Packing specifications】

Type: DIFF Lyse

【Intended use】

This product is used to dissolve RBC and maintain the morphology of the cells needed to be analyzed before blood cell analysis so that facilitate cell sorting or HGB quantification

【Inspection principle】

By dissolving RBC and changing the shape of WBC, DIFF lyse is used to detect WBC classification results.

【Main compositions】

Na₂SO₄, Surfactant, Propylene glycol

【Storage conditions and validity】

This product should be stored between 2°C ~ 30°C, the shelf life of DIFF lyse is 2 years from the date of manufacture. If stored between 15°C ~ 30°C, the validity for the opened reagent is 60 days.

【Application range】

Applicable to Hemia 55 Auto Hematology Analyzers manufactured by Sicuro Bioengineering Ltd.

【Sample requirements】

The collection of samples requires no agglomeration and contamination; the samples should be used in 4 hours.

【Use method】

Restore DIFF lyse to room temperature, open the package, insert the corresponding lyse catheter into the bottle according to the reagent cap and the bottle cap color correspondence, and tighten the cap assembly, and follow the instructions in the instrument to carry out reagent replacement operation. And then blank measurement, before the sample test makes sure the measured value in the instrument manual required blank count range, please refer to the instruction manual.

【Reference value】

Not applicable

【Performance indicators】

This product should be colorless transparent liquid, without precipitation, granules or floccule.
Blank value: WBC≤0.2×10⁹/L, HGB≤1g/L.

【Warning】

1. Do not drink, once the liquid flows into your mouth; Avoid contact with the skins, once get contacted, immediately wash with big amount of water and go to see a doctor immediately.
2. Do not use after the expiry date.
3. If there is a freeze phenomenon, thaw at room temperature, mix well, make sure that the blank test is within range
4. Dispose the waste, and residual reagents in accordance with local regulations and related environmental requirements

The following factors will interfere the tests:

1. The reagents expired or become invalid.
2. Reagents get contaminated by the dust in the air.
3. Samples were not handled cleanly.
4. Mixed or combined with other factories' reagents.

【References】

Refer to Instruction Manual

【Symbols explanation】

Label	Meaning
	Date of manufacture
	In vitro diagnostic medical device
	Manufacturer
	Batch code
	Temperature limit
	Use-by date
	Volume
	Consult instructions for use
	Do not re-use
	Keep away from sunlight
	Contains sufficient for <n>tests
	Catalogue number

【Manufacturer】

 Sicuro Bioengineering Ltd.
Address: Sicuro House, 19 East/ West Rd Port Harcourt Nigeria.
Email: info@sicurobio.com
Website: www.sicurobio.com

【Instruction approved and revised date】

Approved date: April 20th 2024
Revised date: May 7th 2026

【Guarantee and Technical Support】

If the invalid message repeats or needs technical support, please contact Sicuro Customer Service and Support Center.
Contact no: +2348037219600

